DENTURE CLEANER

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DENATURE CLEANER

INSTRUCTIONS FOR USE - EN

Product characteristics: Auxiliary material for use in technical and dental laboratories as well as dental clinics. Liquid – for dissolving calculus and eliminating unpleasant odor from dental dentures used. Powder – for removing stains caused by food or nicotine-containing products.

Intended use: dental dentures, professional cleaning of removable dental dentures.

Contraindications: Do not use the product for patients allergic to any of its ingredients. This product must not be used by patients with a decreased immunity to be ingredients.

Adverse reactions: No specific adverse reactions are known at the moment.

Colour shades: Not applicable

Assortments

Powder:

REF TP300 Denture cleaner Powder 150 g

Liquid:

REF TP301 Denture cleaner Liquid 500 ml

Chemical composition

Powder: amidosulfonic acid and sodium persulfate

Liquid: phosphoric acid (V) and isopropyl alcohol

Instructions for use

Use: The manufacturer recommends that the liquid should be first used to remove calculus, and then powder to remove stains from the dentures. **Denture cleaner Liquid and Denture cleaner Powder** do not form a set and either only liquid or only powder should be used as **caution!** Do not mix **Denture cleaner Liquid and Denture cleaner Powder**.

Liquid: Preparation of calculus removal and cleaning solution:

1. Pour warm water containing **Denture cleaner Liquid** (diluted at 1:1) into a container made of a chemically resistant material (e.g. glass) so that the dentures are fully immersed in the solution.

For hygienisation and quick clean, 10 minutes are sufficient. For better results, longer soaking time may be used.

Caution! To achieve greater efficiency of the cleaning process, place the container with the dentures soaked in the liquid in an ultrasonic cleaning unit.

2. For best results, clean the dentures with a steam gun or brush after removing from the solution.

Caution! If the dentures show a thick calculus layer, repeat the procedure of soaking and cleaning with a steam gun.

3. The remaining solution may be used several times, depending on the degree of staining. If the solution is used again, the dentures must be disinfected (after using the liquid).

4. Rinse the dentures with running water and disinfect before placing them in the patient's mouth.

Powder: Preparation of stain and plaque removing solution:

1. Pour 200 mL of warm water (40° - 60°C) into a container made of a chemically resistant material (e.g. glass) and add 5 g (1 teaspoon) of **Denture cleaner Powder**. The dentures should be fully immersed in the solution.

2. After placing the dentures in the solution, a vigorous reaction of gas formation takes place.

3. For the removal of stains, soak the dentures for 10 minutes. If necessary, the dentures may be left in the solution for up to 15 minutes.

4. Discard the remaining solution.

5. Rinse the dentures with running water and disinfect before placing them in the patient's mouth.

Caution! If the dentures show signs of long-term use, there may be a marginal cleft at the point of contact between the acrylic material and teeth surface. Dark stains due to the absorption of organic substances contained e.g. in coffee or cigarettes will be removed. As a result, white post-oxidation residues may be observed. Since the structural cohesion of acrylic material tends to diminish over time, while "spider's webs" may be observed in some places on the dentures due to the oxygenation of organic inclusions in the dentures structure. To minimise this effect, reduce the time of soaking in the solution removing stains.

Risks and prevention

Information for dental technicians:

Caution! There is a risk of infection transmission. The dentures should be washed and disinfected at all times before placing them in the mouth.

Powder:

H314 Causes severe skin burns and eye damage.

H332 May cause drowsiness or dizziness.

P264 Wash hands thoroughly after handling.

P270 Do not eat, drink or smoke when using this product.

DENATURE CLEANER

INSTRUCTIONS FOR USE - EN

P273 Avoid release to the environment.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P302+P352 IF ON SKIN: Wash with plenty of soap and water.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing.

P310 Immediately call a POISON CENTER or doctor/physician.

P501 Dispose of contents/container to an authorized contractor.

Liquid:

H272 May intensify fire, oxidizer.

H302 Harmful if swallowed.

H315 Causes skin irritation.

H318 Causes serious eye damage.

H412 Harmful to aquatic life with long lasting effects.

P220 Keep/Store away from clothing/combustible materials.

P221 Take any precaution to avoid mixing with combustibles.

P264 Wash hands thoroughly after handling.

P270 Do not eat, drink or smoke when using this product.

P273Avoid release to the environment.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P301+P312 IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell.

P302+P352 IF ON SKIN: Wash with plenty of soap and water.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing.

P332+P313 IF SKIN IRRITATION OCCURS: Get medical advice/attention

P362 Take off contaminated clothing and wash before reuse.

P501 Dispose of contents/container to an authorized contractor.

Powder and liquid:

In case of inhalation:

If breathing, move the victim to fresh air and keep at rest. If not breathing, apply artificial respiration. Seek medical attention.

In case of skin contact:

Wash the affected area immediately with plenty of water and soap. Continue rinsing for 10 minutes. In case of burns or irritation, seek medical attention.

In case of eye contact:

Rinse thoroughly with plenty of water while keeping the eyelids wide open for at least 15 minutes and seek medical attention.

If swallowed:

Never give anything by mouth to an unconscious person. Rinse mouth immediately with water. Give a little water to drink. Do not induce vomiting. Seek medical advice immediately and show this container or label.

Information for dentists:

Caution! There is a risk of infection transmission. The dentures should be washed and disinfected at all times before placing them in the mouth.

Storage and handling

Store in a dry and well-ventilated area, in sealed commercial packaging, at a temperature not exceeding 25°C.

Do not use the product after the expiry date which is stated on the commercial packaging and labels of product ingredients.

Disposal considerations

Dispose of in accordance with the relevant local regulations. Cardboard packaging, labels, instructions – recyclable, return as non-hazardous domestic waste. Liquid and powder packaging – hazardous waste.

Deliver to an authorised provider of waste disposal services. Used working solutions of powder or liquid – hazardous waste. Deliver to an authorised provider of waste disposal services.

Caution! Material for professional use only in technical and dental laboratories as well as dental clinics.

All written and/or verbal instructions and information given during the presentation by the manufacturer are in accordance with the current level of dental techniques and technology. They should be considered applicable information. No information acquired from any third parties (given their different level of authorisation) releases the user from checking if the product is suitable for the intended particular use. Since the manufacturer has no control over the use of the product, the manufacturer's liability only covers the value of the supplied material.

Any comments relating to our products should be sent to our address.