VILLACRYL ORTHO, ORTHO MIX COLOR CONCENTRATE INSTRUCTIONS FOR USE - EN

Product description: Solutions of organic dyes in methyl methacrylate

Classification according to PN-EN ISO 20795-2: Type I.

Intended use: Mobile orthodontic appliances, repair of acrylic elements of orthodontic devices.

Clinical benefits

Correcting minor dental changes. Retaining the corrected dentition following the use of orthodontic devices.

Restoring the proper functioning of orthodontic appliances damaged during use.

Target patient group(s) and intended users

Users

Dental technology professionals – board certified dental technicians providing mobile denture services.

Patients

Persons of different age and sex, mostly children and adolescents, with minor occlusal issues, who are ineligible for treatment using fixed orthodontic appliances.

Persons of different ages and sex, mostly children and adolescents, following the completion of orthodontic treatment using fixed or mobile devices.

Contraindications

Do not use the product in persons with known lack of immunity to allergens, including the product's ingredients. Use other materials that are suitable for the patient.

The product should not be used by dental technicians who are pregnant or breastfeeding.

Product form

50 ml of liquid plus instructions for use.

Composition: methyl methacrylate, ethyl methacrylate, 1,4-butanediol dimethacrylate.

Product compatibility

For use with standard materials for the production of removable braces **Villacryl Ortho**, **Villacryl Ortho Mix** and others based on PMMA.

Safety guidelines

Warnings

H225 Highly flammable liquid and vapor.

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

Liquid sensitive to temperature and UV light. Uncontrolled polymerization may occur.

Precautions

If you have an allergic reaction, stop using the product immediately and consult a dermatology specialist.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

P261 Avoid breathing dust/fume/gas/mist/vapors/spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P302 + P352 IF ON SKIN: Wash with plenty of soap and water.

P501 Dispose of contents/container to a licensed waste disposal service provider.

It is recommended that the following personal protection equipment be used: HDPE gloves for hand protection, a filtering face mask appropriate for liquids for respiratory protection, protective glasses and working clothes.

The use of forced ventilation of rooms and local exhaust ventilation systems for workstations is recommended.

If the product splashes into your eyes, flush your eyes with water. If irritation occurs, seek medical advice.

Do not eat or drink when handling this product. If ingested by accident, seek medical advice immediately.

Do not discharge into drains.

Check that the liquid has retained its properties each time before using the product. If you notice any visible signs of changes in density, cloudiness or discoloration, discontinue the use of the product.

Undesirable effects

Dental technicians

Irritation, redness or symptoms of hypersensitivity, including anaphylactic shock, in case of allergy to any of the product's ingredients.

Patients

Irritation, inflammation and allergic reactions of the mouth due to a residual monomer being washed out of the acrylic plate forming part of the orthodontic appliance.

Storage conditions

Color concentrates sensitive to temperature and UV light. Protect from light and UV radiation.

Store the product in original, tightly sealed containers, in a well-ventilated area, at a temperature of 5-25°C, out of the reach of third parties.

Disposal considerations

Dispose of in accordance with relevant regulatory requirements for the management of waste from business operations.

Cardboard boxes, labels, instructions: non-hazardous segregated municipal waste – paper.

Concentrate packaging – hazardous waste, follow relevant regulations on the management of hazardous waste (storage, transport and disposal) and dispose of hazardous waste via a professional hazardous waste disposal site.

Directions for use

Add 20 ml of Villacryl Ortho liquid to 1 ml (about 20 drops) of Villacryl Ortho color concentrate. Different color saturations can be obtained by increasing or decreasing the amount of concentrate. Different concentrates can be mixed to achieve complementary colors.

Lot number and expiry date – see the information printed on the product labels and packaging.

The expiry date refers to an unopened packaging which has been kept under proper storage conditions. Do not use after the expiry date.

Additional information

Reporting serious incidents: Any serious incidents related to the product should be reported to the manufacturer and the competent authority as required under local regulations.

Information necessary to identify the product and package contents

Reference number, product name, color, pack size, lot number and expiry date – information printed on the product

packaging.

Colors

Villacryl Ortho Villacryl Ortho Mix	Color concentrates
50 ml	yellow, orange, red, raspberry, purple, green, light green, blue.

Additional information

Material for use in dental prosthetic laboratories only.

No information provided by any available means, even during demo presentations, replaces the instructions for use. Users are responsible for checking that the product is suitable for the intended use. The manufacturer is not liable for any damage, including damage to third parties, resulting from non-compliance with the instructions or unsuitability for use. In no case will the manufacturer's liability exceed the value of the products supplied.

MD

For detailed information about all materials supplied by **Everall7**, please contact us at +48 602 701 001.

MD

Any comments relating to our products should be sent to our address.

MD

This symbol represents a medical device as defined in Regulation (EU) 2017/745.

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