Villacryl IT LC Light-curing trays

Product description

Product characteristics Villacryl IT LC is a light-curing acrylic material for making custom impression trays.

Indications for use: making custom impression trays during the preparation of dental prostheses treatment.

Clinical benefits: Reproducing the dynamic conditions of the prosthetic filed (area) in the fabrication of dental prostheses. Obtaining an accurate impression of the prosthetic area in post-implant prosthetics when the use of a standard impression tray does not guarantee the required impression accuracy.

Target group(s) of patients and intended users of the products

Users: Professionals in the dental technology industry – certified dental technicians making dental prostheses. Dentists taking impressions with custom trays.

Patients: Patients requiring prosthetic treatment (removable complete and partial dentures and implant-supported dentures) in situations where the use of a standard impression tray does not guarantee the required impression accuracy.

Contraindications

Do not use the product in people allergic to any of its ingredients.

Do not use the product in patients with a diagnosed allergic sensitivity.

Dental technicians who are pregnant or breastfeeding should not use the product.

Adverse effects

Dental technicians

Irritation, redness or hypersensitivity symptoms, including the occurrence of anaphylactic shock if allergic to any of the product's ingredients.

Patients

Irritation, inflammation and hypersensitivity reactions.

Product form: Plates of pink material at least 2 mm thick. Packed in a black PE box with viscose separators and black PE film.

How supplied: PE box containing 50 plates.

Product composition: bisphenol A epoxide diacrylate, urethane dimethacrylate and diphenyl(2,4,6-trimethylbenzoyl)phosphine oxide.

Compatible products

For use with impression materials and adhesives for impression trays.

Safety instructions

Risks

H317 May cause an allergic skin reaction.

EUH 208 Contains (1-methyl-1,2-ethanedithiol)bis[oxymethyl-2,1-ethanedithiol] diacrylate May produce an allergic reaction.

If powder particles enter the eyes during the processing of an impression tray, mechanical irritation may occur.

Dust generated during product processing may irritate the skin and respiratory tract and cause mechanical irritation to the eyes.

Precautions

P261 Avoid breathing dust/fume/gas/mist/vapours/ spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection/hearing protection/...

P333+P313 If skin irritation or rash occurs: Get medical advice/attention.

Standard protective equipment used in dental laboratories – protective masks, goggles, gloves, and aprons – are sufficient for the safe handling of the product.

Do not eat or smoke while working with the material.

Work in a well-ventilated room. Use ventilation in the workplace.

Information relevant to the patient/user of the product

Dental technicians

If an allergic reaction occurs, stop work immediately and consult an allergist.

In the event of anaphylactic shock, take immediate emergency intervention.

If the material comes into contact with the eyes, rinse them with water and consult a doctor.

Thoroughly wash areas of skin that have come into contact with the product ingredient using soap and water.

In the event of accidental ingestion of a product ingredient, contact a doctor.

Dentist/patient

Do not use the product in patients with a diagnosed allergic sensitivity. Use of the product may cause various hypersensitivity reactions, including anaphylactic shock. In such situations, use a different material, suitable for the patient.

In the event of any allergic reaction to the tray material, immediately discontinue taking the patient's impression.

If a piece of a damaged tray is swallowed when taking impression, send the patient to the doctor. The material is not radiopaque.

Storage conditions

The material is sensitive to temperature and UV radiation, which may cause an uncontrolled polymerisation reaction.

Store the product in the original tightly closed packaging in a dark and well-ventilated room at 5-25 °C out of the reach of unauthorised persons.

Waste disposal

Dispose of in accordance with current business waste management regulations.

Labels, instructions: non-hazardous segregated municipal waste – paper.

PE packaging: non-hazardous segregated municipal waste – plastic.

Uncured material: environmentally hazardous waste. Hand over to an authorised waste collector.

Use impression tray: medical waste. Hand over to an authorised waste collector.

Directions for use (step-by-step instructions)

Preliminary activities:

Read the instructions carefully before using the product.

Each time before reusing the product, check whether the material has retained its properties.

Before making an impression tray, ensure that the patient for whom the impression tray is being made is not allergic to any of the material's ingredients.

Preparation

Coat the plaster model twice with Izo-Sol (gypsum – acrylic) insulating liquid. Wait until the insulating liquid is fully absorbed.

Forming the tray

Form the tray on the plaster model and trim the edges removing excess material. Prepare a tray handle and attach to the formed tray.

Polymerisation

Place the tray on the model in a polymerisation lamp with a radiation range of 380-420 nm for 3-5 minutes. Irradiation of the light-curing mass in a 25–35 W lamp improves the properties of the mass and reduces shrinkage.

Final processing of the completed impression tray

After removing the material from the lamp, wipe the surfaces of the custom tray with alcohol to remove the uncured layer (oxygen inhibition layer). Use the standard method of finishing. During finishing, the material becomes dusty.

Packing and transportation of the impression tray to the dental practice

Secure the completed impression tray against mechanical damage and high temperatures, then hand it over to the dentist.

Hygiene, sterilisation, disinfection

Product

The product plates are intended for single use (one plate is used to make one impression tray). They are not supplied sterile, nor are they required to be sterilised or disinfected before use.

Apply the principles of good laboratory practice in dental laboratories to protect the product from contamination.

Do not eat or smoke while working with the material.

Work in a well-ventilated room. Use ventilation in the workplace.

Information relevant to the patient/user of the product

Dental technicians

If an allergic reaction occurs, stop work immediately and consult an allergist.

In the event of anaphylactic shock, take immediate emergency intervention.

If the material comes into contact with the eyes, rinse them with water and consult a doctor.

Agents based on at least 65% ethyl or propyl alcohol can also be used.

The tray should be sprayed with a disinfectant solution and then gently wiped dry with a paper tissue.

Ready-to-use disposable wipes soaked in a disinfectant solution can also be used.

Additionally, the tray should be washed with warm water and soap.

Additional information

Reporting serious incidents: Any serious incidents involving the products must be reported to the manufacturer and to the competent authority in accordance with local regulations.

Data necessary for the identification of the product and package contents

Reference number, product name, shade, batch number and expiry date – printed on the product packaging.

The expiry date refers to the unopened packaging properly stored.

Do not use the product after the expiry date.

Packaging contents – see table below.

REF	Product description
V330ITLC50	Villacryl IT LC 50 plates in pink

Problem prevention

See: [www.everall7.pl](#)

For details on all Everall7 materials, please call: +48 602 701 001.

Please send any comments on our products to our address.

Information provided in any way, including demonstrations, does not replace reading the instructions for use. Users are required to make sure that the product is suitable for the intended use. The manufacturer shall not be held liable for damage, including to third parties, resulting from a failure to follow the instructions or unsuitability for use. The manufacturer's liability is in any case limited to the value of the products supplied.

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