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Villacryl H Rapid

I. Product description

Villacryl H Plus: Heat curing acrylic resin for dental denture bases
Villacryl H Rapid: Fast heat curing acrylic resin for dental denture bases
Villacryl H Rapid FN: Fast heat curing acrylic resin for dental denture bases
ISO classification: 20795-1; Type I Class 1
Processing parameters: necessary parameters for the proper processing of the materials are given in **Table 1**.
Indications for use
Villacryl H Plus: complete removable dentures (edentulous), partial tissue supported removable dentures, relining of removable prosthesis in dental laboratory
Villacryl H Rapid: complete removable dentures (edentulous), partial tissue supported removable dentures. In case of thick base of the denture, it is recommended to use **Villacryl H Plus**.
Villacryl H Rapid FN: complete removable dentures (edentulous), partial tissue supported removable dentures. In case of thick base of the denture, it is recommended to use **Villacryl H Plus**.
Clinical benefits:
 Restoration of biting and chewing functions lost as a result of partial or complete loss of teeth.
Target group or target groups of patients and intended uses of the devices
Users: Professionals in the dental technology industry - certified dental technicians producing dental prostheses.
Patients: People of different ages and sexes, with partial or complete lack of teeth, eligible for prosthetic treatment with removable dentures, children, adolescents, adults.

Contraindications:
 Do not use the products by people disposed with a lack of resistance to allergenic factors, including the ingredients of the product. In such situations, use a different material suitable for the patient.
 Do not use the product by pregnant women and nursing mothers in the group of dental technicians.

Product form
 Separate a mixture of powder in plastic containers and liquid in bottles. The product is available in the form of a set: powder + liquid or separately as a powder and a liquid.

Chemical composition
Villacryl H Plus: powder: polymethyl methacrylate, pigments, viscose fibres *
 Liquid: methyl methacrylate, ethyl methacrylate, cross-linking agent, stabilizer.
Villacryl H Rapid: powder: polymethyl methacrylate, catalyst, pigments, viscose fibres *
 Liquid: methyl methacrylate, ethyl methacrylate, cross-linking agent, stabilizer.
Villacryl H Rapid FN: powder: polymethyl methacrylate, pigments, viscose fibres *
 Liquid: methyl methacrylate, ethyl methacrylate, cross-linking agent, stabilizer.

Product compatibility
 For use with acrylic artificial teeth.
Safety instructions
Warnings:
 H225 Highly flammable liquid and vapour.
 H315 Causes skin irritation.
 H317 May cause an allergic skin reaction.
 H335 May cause respiratory irritation.

Precautions:
 The liquid is sensitive to temperature and UV radiation. An uncontrolled polymerization reaction may occur.
Cooling and deflasking
 The correct parameters of the cooling process after polymerization for the material are given in **Table 1**.
Standard method of polymerization: Take the frame with flask out of the polymerization device and cool it down to room temperature. When fully cooled open the flask and remove the plaster.
Preparation of the mold: Leave the frame with the flask in the polymerization device until it cools down to the ambient temperature spontaneously. Take out from water, carefully open and remove the plaster.
Residual monomer level reduction: Respect the following for the material: recommended mixing ratio, mixing temperature and time of mixing components and parameters of the polymerization process.
 Reducing the residual monomer level can be achieved by extending the polymerization time. Before handing over the denture to the dentist, prosthesis should be soaked in water at room temperature for 48 hours, this operation can also be performed by the patient after the dentist has handed over the denture.
Final processing: Use standard method of treatment. Polishing with **Everall7 Pumeks and Pasta polerska**.
 The denture base should be thoroughly polished. Scratches are a place of dirt accumulation. Do not overheat the material during processing and polishing - permanent changes in the structure of the material may occur, leading to an increase in residual monomer level in the finished prosthesis.

Corrections and repairs of a damaged denture: Corrections and repairs of a damaged denture can be performed with Villacryl SP or various preparations for denting and caring for dental prostheses that do not contain abrasives. It is recommended to wash the dentures from time to time in a vibration washbasin with a special cleaning liquid for dentures.
 When not in use, wash the denture and store it in a special denture case.
 Do not leave the prosthesis in the sun, near sources of heat and open flames.
 The prosthesis is not resistant to long-term effects of alcoholic beverages.
 The denture may change color or lose its original physical properties when exposed to chemicals or if it is misused.
 During use, the prosthesis should be periodically inspected in dental office.
 Report any damage of the denture to the dentist. Repairs may be only performed by a dental technician.
 If a piece of damaged denture is swallowed, consult a doctor immediately. The material after absorption is not visible on X-ray pictures. It is recommended to relime or rebase the denture plate in the event of the disappearance of the alveolar process.

Side effects:
Dental technicians
 Irritation, redness or symptoms of hypersensitivity, including the occurrence of anaphylactic shock, in case of allergy to any of the product ingredients.
Patients
 Irritation, inflammation and allergic reactions caused by residual monomer washed out of the denture.

Storage conditions:
 Product is sensitive to temperature and UV radiation. Protect against light and UV radiation.
 Store the product components in their original, tightly closed containers, in a dark and well-ventilated room, at a temperature of 5-25 °C, out of the reach of third parties.
Waste disposal
 Dispose in accordance with applicable regulations of economic activity waste management.
Villacryl H Rapid FN: Fast heat curing acrylic resin for dental denture bases
ISO classification: 20795-1; Type I Class 1
Processing parameters: necessary parameters for the proper processing of the materials are given in **Table 1**.
Indications for use
Villacryl H Plus: complete removable dentures (edentulous), partial tissue supported removable dentures, relining of removable prosthesis in dental laboratory
Villacryl H Rapid: complete removable dentures (edentulous), partial tissue supported removable dentures. In case of thick base of the denture, it is recommended to use **Villacryl H Plus**.
Villacryl H Rapid FN: complete removable dentures (edentulous), partial tissue supported removable dentures. In case of thick base of the denture, it is recommended to use **Villacryl H Plus**.
Clinical benefits:
 Restoration of biting and chewing functions lost as a result of partial or complete loss of teeth.
Target group or target groups of patients and intended uses of the devices
Users: Professionals in the dental technology industry - certified dental technicians producing dental prostheses.
Patients: People of different ages and sexes, with partial or complete lack of teeth, eligible for prosthetic treatment with removable dentures, children, adolescents, adults.

Contraindications:
 Do not use the products by people disposed with a lack of resistance to allergenic factors, including the ingredients of the product. In such situations, use a different material suitable for the patient.
 Do not use the product by pregnant women and nursing mothers in the group of dental technicians.

Product form
 Separate a mixture of powder in plastic containers and liquid in bottles. The product is available in the form of a set: powder + liquid or separately as a powder and a liquid.

Chemical composition
Villacryl H Plus: powder: polymethyl methacrylate, pigments, viscose fibres *
 Liquid: methyl methacrylate, ethyl methacrylate, cross-linking agent, stabilizer.
Villacryl H Rapid: powder: polymethyl methacrylate, catalyst, pigments, viscose fibres *
 Liquid: methyl methacrylate, ethyl methacrylate, cross-linking agent, stabilizer.
Villacryl H Rapid FN: powder: polymethyl methacrylate, pigments, viscose fibres *
 Liquid: methyl methacrylate, ethyl methacrylate, cross-linking agent, stabilizer.

Product compatibility
 For use with acrylic artificial teeth.
Safety instructions
Warnings:
 H225 Highly flammable liquid and vapour.
 H315 Causes skin irritation.
 H317 May cause an allergic skin reaction.
 H335 May cause respiratory irritation.

Precautions:
 The liquid is sensitive to temperature and UV radiation. An uncontrolled polymerization reaction may occur.
Cooling and deflasking
 The correct parameters of the cooling process after polymerization for the material are given in **Table 1**.
Standard method of polymerization: Take the frame with flask out of the polymerization device and cool it down to room temperature. When fully cooled open the flask and remove the plaster.
Preparation of the mold: Leave the frame with the flask in the polymerization device until it cools down to the ambient temperature spontaneously. Take out from water, carefully open and remove the plaster.
Residual monomer level reduction: Respect the following for the material: recommended mixing ratio, mixing temperature and time of mixing components and parameters of the polymerization process.
 Reducing the residual monomer level can be achieved by extending the polymerization time. Before handing over the denture to the dentist, prosthesis should be soaked in water at room temperature for 48 hours, this operation can also be performed by the patient after the dentist has handed over the denture.
Final processing: Use standard method of treatment. Polishing with **Everall7 Pumeks and Pasta polerska**.
 The denture base should be thoroughly polished. Scratches are a place of dirt accumulation. Do not overheat the material during processing and polishing - permanent changes in the structure of the material may occur, leading to an increase in residual monomer level in the finished prosthesis.

Corrections and repairs of a damaged denture: Corrections and repairs of a damaged denture can be performed with Villacryl SP or various preparations for denting and caring for dental prostheses that do not contain abrasives. It is recommended to wash the dentures from time to time in a vibration washbasin with a special cleaning liquid for dentures.
 When not in use, wash the denture and store it in a special denture case.
 Do not leave the prosthesis in the sun, near sources of heat and open flames.
 The prosthesis is not resistant to long-term effects of alcoholic beverages.
 The denture may change color or lose its original physical properties when exposed to chemicals or if it is misused.
 During use, the prosthesis should be periodically inspected in dental office.
 Report any damage of the denture to the dentist. Repairs may be only performed by a dental technician.
 If a piece of damaged denture is swallowed, consult a doctor immediately. The material after absorption is not visible on X-ray pictures. It is recommended to relime or rebase the denture plate in the event of the disappearance of the alveolar process.

Side effects:
Dental technicians
 Irritation, redness or symptoms of hypersensitivity, including the occurrence of anaphylactic shock, in case of allergy to any of the product ingredients.
Patients
 Irritation, inflammation and allergic reactions caused by residual monomer washed out of the denture.

Batch number and expiry date: see imprint on product labels and packaging. The expiry date refers to properly stored unopened packaging. Do not use the product after the expiry date.
Additional information
Reporting of serious incidents: Any serious incidents related to the products must be reported to the manufacturer and the competent authority in accordance with local regulations.
Data necessary to identify the product and the contents of the package
 Reference number, product name, color, package size, batch number and expiry date - imprint on the product packaging.
 Package content - see table below.
Assortment*
Packaging

	Powder + liquid kit	Powder	Liquid
Villacryl H Plus	750 g powder + 400 ml liquid 750 g powder + 2 x 400 ml liquid 300 g powder + 150 ml liquid	750 g, 300 g, 30 g, 24 g, 2 kg, 4 kg, 20 kg	400 ml, 150 ml, 12 ml, 10ml, 1 l
Villacryl H Rapid	750 g powder + 400 ml liquid	750 g, 24 g, 2 kg, 4kg	400 ml, 10 ml, 1 l
Villacryl H Rapid FN	750 g powder + 400 ml liquid	24 g	400 ml, 10 ml

*Information on the availability of the assortment is provided by the distributor.
Processings parameters (1)

Processing parameters	Villacryl H Plus	Villacryl H Rapid	Villacryl H Rapid FN
1 Mixing ratio powder/liquid	24 g / 10 g (10,5 ml)	24 g / 10 g (10,5 ml)	23 g / 10 g (10,5 ml)
2 Setting time (23±2°C)	20-25'	8-10'	8-10'
3 Suitability time for use	25'-30'	20'	20'
4 Polymerization: Standard:	30'-60'' → 100°C 30'-100°C	10'-80'' → 100°C, 20'-100°C	10'-80'' → 100°C, 20'-100°C
Prolonged:	Cold water → slowly heating to 100°C → keep boiling at a minimum 30'	Cold water → slowly heating to 100°C → keep boiling at a minimum 30'	20°C → slowly heating to 100°C → keep boiling at a minimum 30'
5 Cooling: Standard:	Spontaneously in the air by 30' → 25°C	Spontaneously in the air by 20' → 30°C	Spontaneously in the air by 20' → 30°C
Prolonged:	Leave in the polymerizer to spontaneously cool down to ambient temperature.	Leave in the polymerizer to spontaneously cool down to ambient temperature.	Leave in the polymerizer to spontaneously cool down to ambient temperature.

Problem prevention (2)
 See website: www.everall7.pl

Please send any comments regarding our products to our address.
 Information provided in any way, even during the demonstration, does not invalidate the instructions for use. Users are responsible for checking that the product is suitable for the intended use. The manufacturer is not liable for damages, including to third parties, resulting from non-compliance with the instructions or for use. The manufacturer's liability is in any case limited to the value of the products delivered.

MD The symbol means a medical device as defined in EU Regulation 2017/745.
INSTRUKCJA UŻYCIA - PL

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