SECRET AGENT	INSTRUCTIONS FOR USE - EN
---------------------	----------------------------------

Secret Agent. Release agent for printing 3D models

Product description

Product characteristics: Release agent for separating the surface of a 3D resin model from acrylic.

Indications for use: Ancillary work in dental technology laboratories.

Separation of cold-curing acrylic from a 3D-printed resin model, when fabricating plates for orthodontic appliances or denture repairs.

Contraindications

Do not use in people who have allergic reactions to ethyl alcohol.

Product form

Secret Agent is available in two independent variants for different applications:

Powder – used to cure models after post-processing.

Liquid – a standalone isolator that separates the acrylic resin from the printed model.

Product composition

Liquid: ethyl alcohol, dye

Powder: sodium sulphate, CTMABr

Safety instructions

Warnings

Liquid

H225 Highly flammable liquid and vapour.

P210 Keep away from heat, hot surfaces, sparks, open flames and other sources of ignition.

No smoking.

P233 Keep container tightly closed.

P370+P378 In case of fire: Use a powder extinguisher, sand, or carbon dioxide to extinguish.

P403+P235 Store in a well-ventilated place. Store in a cool place.

P501 Dispose of contents/container to an approved waste disposal plant or return to supplier.

Powder

H400 Very toxic to aquatic organisms.

H412 Harmful to aquatic life with long-lasting effects.

P273 Avoid release to the environment.

P501 Dispose of contents/container to an approved waste disposal plant or return to supplier.

Precautions

In the event of any allergic reaction, stop work immediately and consult a dermatologist.

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources.

No smoking.

P261 Avoid breathing dust/fume/gas/mist/vapours/spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Contact with a doctor is advisable if irritation occurs.

P312 Call a Poison Centre or doctor/physician if you feel unwell.

P403+P235 Store in a well-ventilated place. Keep container tightly closed.

P235 Keep cool.

P501 Dispose of contents/container to an approved waste disposal plant.

Storage conditions of the product

The liquid contains flammable substances. Keep away from open flames and radiant heat. Store product components in original, tightly closed packaging, in a dark and well-ventilated room, at 5–25°C, out of the reach of unauthorised persons.

Waste disposal

Dispose of in accordance with current business waste management regulations. Cardboard packaging, labels, instructions: non-hazardous segregated municipal waste – paper.

Powder: hazardous waste, follow hazardous waste management regulations (storage, transport and disposal) and hand over to a professional hazardous waste collector. Powder packaging: non-hazardous segregated municipal waste – plastic. Liquid and liquid packaging – hazardous waste, follow hazardous waste management regulations (storage, transport and disposal) and hand over to a professional hazardous waste collector.

Directions for use

Thoroughly wash the printed model with (isopropyl) alcohol in an ultrasonic cleaner and cure in a polymerisation lamp as recommended by the resin manufacturer.

The processes of thorough washing and curing in a UV lamp are crucial for the release agent to work properly and separate the acrylic resin from the 3D-printed model. The process of curing and isolation with this material enables seamless curing of type-A silicones for mock-ups and flow injections.

The manufacturer recommends using **Secret Agent Powder** immediately after printing, rinsing with alcohol, and curing in a UV lamp. Using the preparation ensures final curing of models, effective removal of resin residue, and improved surface and mechanical parameters of the print.

Measure 2 level teaspoons – 9 g powder per 150 ml – of **Secret Agent Powder** and dissolve in water at 40°-50°C. Place the printed model, previously washed with isopropyl alcohol, in

the solution for 5-10 minutes. After this time, rinse the model under running water and dry. Using a brush, apply a small amount of **Secret Agent Liquid** to the clean surface of the 3D-printed model prepared as above, spreading it evenly over the entire surface of the model. Wait 5–6 minutes (at 23°C) until the solvent in the material has evaporated. Then check whether a thin layer, which does not stick to the finger, has formed on the model. The acrylic mass can be applied to the surface prepared as above and cured under the conditions specified by the acrylic resin manufacturer. Once polymerisation is complete and the material has cooled, remove the acrylic component from the model.

The release agent forms a very thin layer and is also water-soluble, so it can be easily removed from both the 3D resin model and the cured acrylic.

NOTE: Use only for acrylic polymerisation up to 65°C, not suitable as a release agent for polymerised thermally acrylics cured by boiling in water.

Batch number and expiry date – see imprints on product labels and packaging.

The expiry date refers to the unopened packaging properly stored. Do not use the product after the expiry date.

Additional information

Reporting serious incidents: Any serious incidents related to the product should be reported to the manufacturer and the competent authority in accordance with local regulations.

Data necessary for the identification of the product and package contents

Reference number, product name, packaging size, batch number and expiry date – see imprint on the product packaging.

Available packaging

Kit (powder/liquid)	40g + 2x10 ml
Powder	300g
Liquid	150 ml

Liquid

Prevention of problems

See: www.everall7.pl

Please send any comments on our products to our address.

For details on all Everall7 materials, please call: +48 602 701 001

Information provided in any way, even during demonstrations, does not invalidate the instructions for use. Users are required to make sure that the product is suitable for the intended use. The manufacturer shall not be held liable for damage, including to third parties, resulting from a failure to follow the instructions or unsuitability for use. The manufacturer's liability is in any case limited to the value of the products supplied.

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