## VILLACRYL ACRYLIC STAINS Instruction for use EN

I. **Product description**

**Villacryl Acrylic Stains:** acrylic denture characterization kit

ISO classification 20795-1

**Processing parameters:** necessary parameters for the proper processing of the materials are given in Table 1.

**Indications for use**

**Villacryl Stains:** individual characterization of removable acrylic dentures

**Clinical benefits:**

Restoring biting and chewing functions lost as result of partial or complete loss of teeth, by making a removable acrylic denture.

**Target group or target groups of patients and intended users of the device:**

**Users:** Professionals in the dental technology industry - certified dental technicians producing dental prostheses.

**Patients:** People of different ages and sexes, denture users. Uses of dentures, children, adolescents, adults.

**Contraindications**

Do not use the products by people diagnosed with a lack of resistance to allergenic factors, including the ingredients of the product. In such situations, use a different material suitable for the patient.

Do not use the product by pregnant women and nursing mothers in the group of dental technicians.

**Product form**

A mixture of powder in plastic containers.

The product is available in the form of a set: 8 powders in different colors.

**Chemical composition**

Powder: polymethyl methacrylate (PMMA), pigments, viscose fibers\*

\* Depending on the color variety, the composition of the product may slightly differ in terms of quantity and quality

**Product compatibility**

For use with acrylic artificial teeth and acrylic dentures.

**Safety instructions**

**Warnings**

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

The ingredients of the product and the dust generated during the fabrication of dentures may cause mechanical eye irritation.

**Precautions**

In case of any allergic reaction, stop working immediately and consult a dermatologist.

P261 Avoid dust/fume/gas/mist/vapours/spray.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P302+P352 IF ON SKIN: Wash with plenty of water

P501 Dispose of the contents/container to an authorized consignee of the waste.

It is recommended to use gloves made of HDPE polyethylene for hand protection, a filtering fluid-compatible face mask for respiratory protection, goggles and workwear.

It is advisable to use forced ventilation of rooms and workplace exhaust systems.

In the event of contact with the product ingredient or dust with the eyes, rinse them with water. Contact a doctor is recommended in case of irritation.

Do not eat or drink while working with the material. In the event of accidental ingestion of an ingredient of the product, consult a doctor immediately.

**Information to be provided to patients / prosthesis users:**

To reduce the residual monomer, it is advisable during the first three days, to soak the new prosthesis in water when not in use.

If any symptoms of an allergic reaction or irritation occur, discontinue using the prosthesis immediately and consult a doctor.

Strong oxidizing compounds contained in cleaning agents and mouthwashes may change the color of the prosthesis after polymerization. After each meal, the prosthesis should be washed with a soft brush under cool, running water.

For cleaning, unscented soap can be used, various preparations for cleaning and caring for dental prostheses that do not contain abrasives. It is recommended to wash the dentures from time to time in a vibration washer filled with a special cleaning liquid for dentures.

The material is not resistant to long-term effects of alcoholic beverages.

The denture material may change color or lose its original physical properties when exposed to chemicals or if it is misused.

Report any damage of the denture to the dentist. Repairs may be only performed by a dental technician. If a piece of damaged dentures is swallowed, consult a doctor immediately. The material after polymerization is not visible on X-ray pictures.

It is recommended to relin or rebase the denture base in the event of the disappearance of the alveolar process.

**Side effects**

**Dental technicians**

Irritation, redness or symptoms of hypersensitivity, including the occurrence of anaphylactic shock, in case of allergy to any of the product ingredients.

**Patients**

Irritation, inflammation and allergic reactions caused by residual monomer washed out of the denture.

**Storage conditions**

Store the product components in their original, tightly closed containers, in a dark and well-ventilated room, at a temperature of 5-25 ° C, out of the reach of third parties.

**Waste disposal**

Dispose in accordance with applicable regulations of economic activity waste management.

Cardboard packages, labels, instructions: non-hazardous segregated municipal waste - paper.

Powder: non-hazardous mixed municipal waste

Powder packaging: non-hazardous segregated municipal waste - plastic

**How to use (Step by step instructions)**

**Initial steps:**

Read the instructions carefully before using the product.

Make sure the patient for whom the prosthesis is made has not been diagnosed with the lack of resistance to allergenic factors.

This set consists of 8 PMMA-based powder colors for individual characterization of acrylic dentures.

This material allows for individual characterization of both the denture base and acrylic teeth. Available colors: yellow or brown, allow for visual extension of tooth necks or characterization of existing teeth (e.g., cracks in the enamel structure, visible dentin in worn teeth). Intense blue, dark red, or purple colors can be used to characterize mucosal surfaces.

The material can be used for both thermal polymerization (boiling a new denture) and low-temperature pressure-curing.

**Thermal polymerization**

After boiling out the wax, open the polymerization flask, and thoroughly insulate the plaster surface with an acrylic- plaster insulator (Izo-Sol), wait a few minutes to allow the insulator to fully absorb into the insulated surface and the plaster to reach room temperature. When using technical silicone to protect acrylic teeth, wipe the surface of the silicone material twice with a thermal polymerization monomer, such as Villacryl H Plus. This allows the silicone material to absorb the monomer and prevents the acrylic layer applied to it from drying out. In a ceramic or glass container, mix the appropriate amount of material for one denture, such as Villacryl H Plus or Villacryl H Rapid, in a ratio of 2.4g to 1g powder/thermal polymerization monomer. Approximately 3-4 minutes after combining the powder and liquid (thread phase), apply some of the prepared material to the plaster surface, starting from the tooth surface and gradually applying it upwards, so that the applied acrylic layer is 2-5mm thick.

In another container, mix a small amount of color enhancer and Villacryl H Plus monomer.

Apply a small amount of colorant to the previously applied Villacryl H Plus or Villacryl H Rapid acrylic material using a brush or inlay in the area where you want to achieve a color change. It is also possible to apply the material using the "salt and pepper technique" similar to making removable orthodontic appliances.

Begin with the colorant by staining the marginal gingiva (dark red), and the alveolar mucosa (light pink). Yellow or brown colors are placed around the necks of the tooth.

Important: Where different acrylics meet, they must be mixed together to avoid sudden color changes.

It is advisable to use forced ventilation of rooms and workplace exhaust systems.

In the event of contact with the product ingredient or dust with the eyes, rinse them with water. Contact a doctor is recommended in case of irritation.

Do not eat or drink while working with the material. In the event of accidental ingestion of an ingredient of the product, consult a doctor immediately.

**Information to be provided to patients / prosthesis users:**

To reduce the residual monomer, it is advisable during the first three days, to soak the new prosthesis in water when not in use.

If any symptoms of an allergic reaction or irritation occur, discontinue using the prosthesis immediately and consult a doctor.

Strong oxidizing compounds contained in cleaning agents and mouthwashes may change the color of the prosthesis after polymerization. After each meal, the prosthesis should be washed with a soft brush under cool, running water.

For cleaning, unscented soap can be used, various preparations for cleaning and caring for dental prostheses that do not contain abrasives. It is recommended to wash the dentures from time to time in a vibration washer filled with a special cleaning liquid for dentures.

The material is not resistant to long-term effects of alcoholic beverages. The denture material may change color or lose its original physical properties when exposed to chemicals or if it is misused.

Report any damage of the denture to the dentist. Repairs may be only performed by a dental technician. If a piece of damaged dentures is swallowed, consult a doctor immediately. The material after polymerization is not visible on X-ray pictures.

It is recommended to relin or rebase the denture base in the event of the disappearance of the alveolar process.

The denture should be thoroughly polished. Scratches are a place of dirt accumulation. Do not overheat the material during processing and polishing - permanent changes in the structure of the material may occur, leading to an increase in residual monomer level in the finished prosthesis.

**Packaging and transport of the denture to the dentist's office:** The denture should be protected against mechanical damage and high temperature and handed over to the dentist.

**Hygiene, sterilization, disinfection**

**Product**

The product is intended for multiple use. It is not supplied sterile and does not require sterilization or disinfection prior to use. To protect the product against contamination, apply the principles of good laboratory practices in dental laboratories.

**The prosthesis is made of materials.**

There is a risk of transmission of infection. The repaired prosthesis should be cleaned and disinfected by the dentist before being replaced in the patient's mouth.

For the disinfection of dentures made of materials, it is recommended to use disinfectants for disinfecting dental prostheses, following the procedures recommended by the manufacturer.

It is also possible to use agents based on at least 65% ethyl or propyl alcohol. The denture should be sprayed with a disinfectant solution, and then gently wiped dry with paper tissue. You can also use ready-to-use disposable wipes soaked in a disinfecting solution. Additionally, the denture should be washed with warm soapy water. Prepare the appropriate information to the dentist.

**Batch number and expiry date** - see imprint on product labels and packaging.

The expiry date refers to properly stored unopened packaging. Do not use the product after the expiry date.

**Additional information**

**Reporting of serious incidents:** Any serious incidents related to the products must be reported to the manufacturer and the competent authority in accordance with local regulations.

**Data necessary to identify the product and the contents of the package**

Reference number, product name, color, package size, batch number and expiry date - imprint on the product packaging.

Package content - see table below.

**Assortment\***

**Color shades**

Villacryl Acrylic Stains	
10 g	75 g
White	Transparent Pink
Veined Pink	Light Pink
Yellow	Dark Pink
Brown	
Blue	
Red	
Dark Red	

Powder set	
<b>Villacryl Acrylic Stains</b>	3 x 75 g + 8 x 10 g powders with different colors

\* Information on the availability of the assortment is provided by the distributor.

**Application parameters**

Application parameters	Villacryl Acrylic Stains
1	Mixing ratio powder/liquid 2 g / 1 ml
2	Gel time (23±2°C) ~ 2-3'
4	Polymerization: Thermal 30 min 60°C>100°C, 60 min in 100 °C Water polymerization: Cold Cure min 20 min 60°C, 2 bars

**Problem prevention**

See website: **www.everall7.pl**

*Please send any comments regarding our products to our address.*

Information provided in any way, even during the demonstration, does not invalidate the instructions for use. Users are responsible for checking that the product is suitable for the intended use. The manufacturer is not liable for damages, including third parties, resulting from non-compliance with the instructions or non-use. The manufacturer's liability is in any case limited to the value of the products delivered.

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